

April 2023

Trima Accel® Automated Blood Collection System Software Version 7.1 Release

Dear Valued Customer,

Terumo Blood and Cell Technologies is dedicated to providing you with the highest quality support and remains committed to communicating information regarding our products. This letter is to notify you of the release of Trima Accel software version 7.1.

The version 7.1 software is functionally and operationally equivalent to, and has the same user interface as, a previous software version, 7.0.4. There is no change to the collection procedure, products collected, or operator interaction with the system. The new software is expected to be available in July 2023, subject to regulatory approval in your country.

The major enhancements of version 7.1 are:

- Added capability to support future Trima Accel Non DEHP disposable set releases.
- Added capability to scan lot number and expiration barcodes on either the product bag labels or the barcode located on the tubing set cover.
- Added a firewall to enhance cybersecurity by preventing unauthorized access.
- Added the capability for air removal to be configured Off for non-platelet-capable sets (plasma-only and plasma, RBC sets). This reduces the load time for non-platelet sets. Air removal remains as an option for all sets in the configuration screen.

Other known issues addressed in this release include:

- Improvement of the plasma overcollection alarm. A level sensor alert and a notification have been added to allow the collection to continue, instead of forcing End run or Rinseback as with the current alarm. See Attachment A for screen shots and more details of the new alerts.
- Improvement in the WBC flagging rate for PAS/PPC collections related to LRS chamber monitoring for high concentration in the chamber.

We appreciate your continued support and look forward to serving you. If you have any questions, please visit terumobct.com/contact-us for complete contact information.

Sincerely,



Scott Santi
Global Platform Leader, Global Blood Solutions

Terumo BCT, Inc.
10811 West Collins Ave.
Lakewood, Colorado 80215-4440
USA
USA Phone: 1.877.339.4228
Phone: +1.303.231.4357
Fax: +1.303.542.5215

Terumo BCT Europe N.V.
Europe, Middle East and Africa
Ikaroslaan 41
1930 Zaventem
Belgium
Phone: +32.2.715.0590
Fax: +32.2.721.0770


Terumo BCT Asia Pte. Ltd.
89 Science Park Drive
#04-25 (Lobby B)
The Rutherford
Singapore 118261
Phone: +65.6715.3778
Fax: +65.6774.1419

Terumo BCT Latin America S.A
La Pampa 1517-12th Floor
C1428DZE
Buenos Aires
Argentina
Phone: +54.11.5530.5200
Fax: +54.11.5530.5201

Terumo BCT Japan, Inc.
Tokyo Opera City Tower 49F,
3-20-2, Nishi-Shinjuku,
Shinjuku-ku, Tokyo 163-1450,
Japan
Phone: +81.3.6743.7890
Fax: +81.3.6743.9800

Attachment A: New Alerts With Software Version 7.1

Level sensor error alert: When the system detects a possible plasma volume overcollection, the level sensor error alert is displayed. The system alerts the operator to verify the tubing set is loaded correctly. If it is loaded correctly, the operator can choose to continue or end the procedure.

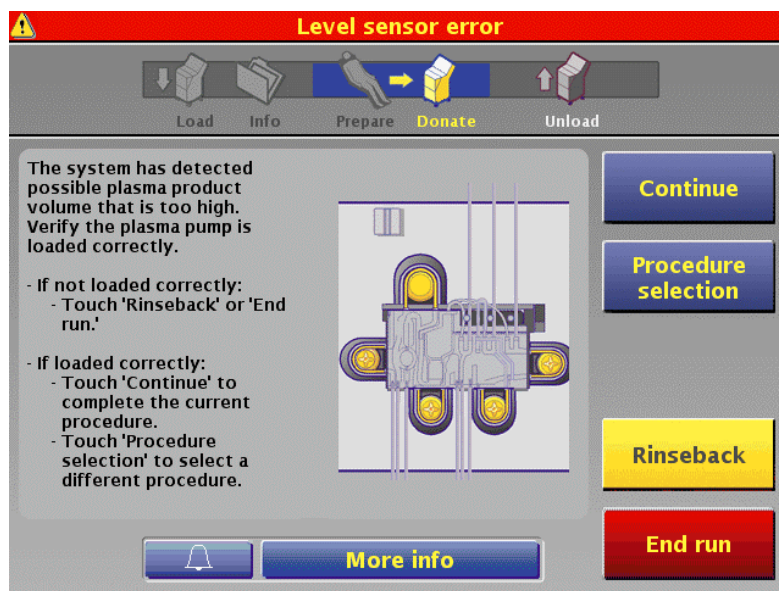

Plasma product volume too high alert: If the operator continues with the procedure from the Level sensor error alert and the system continues to detect plasma overcollection, the notification is displayed. Plasma collection ends, and the system continues collecting the remaining products and then transitions to rinseback.

